

AIR TEMP ADVANTAGE

Lumbar and Abdominal Support

With the advantage of interchangeable air, cold or heat therapy
Supports the muscles of the lower back and abdomen

Helps reduce muscular back discomfort and pain

The ATA encourages proper back alignment

- Pockets rotate off the centre of the back to insert air chambers - Two inflatable air chambers included
- Flexible neoprene lining for warmth and comfort
- Adjustable elastic side straps offer added support when lifting
- Plastic 3/4" support stays with non-slip strips
- Mesh back ensures breathability
- Adjustable hook and loop closure




1. Inflate the air chambers by squeezing the hole at bottom to desired stiffness. Fold bottom up and slide into pocket with folded side first. Hook fastener to secure pocket in centre position. If unhooked, pocket will move to side position when fastening straps. If over time bladder softens, squeeze hole to re-supply air.



2. Loosen elastic straps. Position belt just below the navel. Comfortably attach front closure.

3. Before you lift or when additional support is needed, grasp the ends of the elastic straps, pull around to the front and attach hook closures. Secure elastic straps so that you are comfortable. After lifting, loosen straps and re-attach with less tension to some point further back on your side.

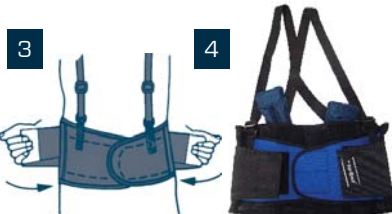

4. When properly fitted, belt should appear as shown. Attach suspenders at the waist with hook and loop closure. Suspenders sold separately, part #SUSPENDERS.

Part#	Size	Waist	
ATAS	Small	24" - 32"	61 - 81 cm
ATAM	Medium	32" - 37"	81 - 94 cm
ATAL	Large	36" - 42"	91 - 106 cm
ATAXL	XLarge	41" - 48"	104 - 122 cm
ATAXXL	XXLarge	46" - 52"	117 - 132 cm
ATAXXXL	XXXLarge	53" - 58"	134 - 147 cm

Sold per each.

SIZING: Use waist measurement or add 2" if belt is worn over clothing.

WASHING: Hand wash in cold water and soft detergent. Do not bleach, do not dry by machine, do not dry clean.

WEIGHT: 0.30 lbs each. 10 per case.

Innovative Protection

ISO 9001:2008



www.impacto.ca

E-mail: impacto@impacto.ca

N. America 888-232-0031

Free phone UK 0800 0280 243

Tel 613- 966-0062 Fax 613-966-0067

Distributed by,